



Designed for Neonatal Nurses ...with Neonatal Nurses

Designed with more than 10 years of MediPurpose's proven product development expertise and with extensive input and validation from neonatal nurses, babyLance™ safety heelsticks are **reliable, consistent, safe and easy to use!**

Reliability: Ultra-Smooth Incisions Minimizes Trauma to Infants' Heels

babyLance's proprietary internal spring design activates its blade's swift pendulum action, which delivers an **ultra-smooth incision** that complies with CLSI LA4-A5 guidelines to **reduce trauma** to an infant's delicate subcutaneous tissues.

Consistency: Accurate Incisions Reduce Need for Multiple Heelsticks

babyLance's ultra-smooth and accurate incisions **consistently produce preferred blood flow volumes...**which reduces the need for multiple heelstick procedures.

Ergonomics: Numerous Features Promote Safety and Ease of Use

babyLance's **intuitive single-use design** makes it one of the safest and easiest heelsticks to use. Featuring the industry's most **effortless activation trigger**, its unique housing provides visual guides for **accuracy** and dimples for a **secure grip**.



Four Easy Steps



1 Select an incision site on the flat bottom surface of the heel, then clean the area.



2 Remove the Trigger Lock, but do not pull back the Trigger until ready for use.



3 Align the Blade Slot's visual marking with the incision site and pull the Trigger back with your index finger. Discard.



4 Gently wipe away the first droplet of blood, then collect the desired quantity. That's it!

babyLance™	Product Code	Depth	Body/Trigger Colors
Micropreemie	BLM	.60 mm	Yellow/Lime Green
Preemie	BLP	.85 mm	Pink/White
Newborn	BLN	1.0 mm	Blue/Green
Toddler	BLT	1.4 mm	Orange/Gray

Americas

3883 Rogers Bridge Road NW
Suite 501
Duluth, GA 30097
Tel: +1 770 419 0400

Asia

15 Hoe Chiang Road #12-02
Tower Fifteen
Singapore 089316
Tel: +65 63454500

Europe

Tel: +44 330 122 2630

MediPurpose™

medipurpose.com/babylance