## Gem II Battery Powered Ring Cutter

# Cleaning, Disinfecting & Autoclaving Chart

#### √ Permissible

#### X Not Permissible

Component	Cleaning Neutral Detergent	Disinfecting Alco-wipe	Manual Washing Sodium Hypochlorite	Autoclave
GEM II Hand Piece	٧	٧	x	х
GEM II Finger Guard	٧	٧	٧	<b>√</b> *
GEM II Cutting Discs	٧	٧	<b>√</b> **	х

<sup>\*</sup> See instructions below

#### NB:

#### **Autoclaving of Finger Guard:**

The Finger Guard consists of two components. An Acetal block and steel guard bonded by two Philips head screws. The steel guard can be autoclaved. The acetal block would distort if autoclaved and can be disinfected in solutions not exceeding 80°c.

### **Cutting Discs:**

Manual washing temperature should not exceed 90°. In view of the relatively low cost of replacement the discs should be disposed of and replaced in cases of serious contamination.

<sup>\*\*</sup> Temperature not to exceed 90°