nitrile  
examination gloves  
ultra comfort

INSTRUCTIONS FOR USE

ENGLISH

Product Description
Klinion Powder-Free Nitrile Examination Gloves are disposable gloves made of nitrile butadiene rubber that is intended to wear on the hand for medical purposes to provide a barrier against potentially infectious materials and other contaminants.

- Directions for use
1. Gloves are packed in dispenser which is suitable for transport. Keep the gloves in the box when not in use and keep away from fluorescent- and sunlight.
2. Before usage, inspect the gloves for any defects or imperfections.
3. Usage: For single use only. If re-used: Extremely high risk of cross-contamination...

Performance and Limitation of Use

This product had been tested in accordance with EN ISO 374-1:2016 + A1:2018, EN ISO 374-2:2019, EN 374-4:2019, EN 16523-1:2015+A1:2018. This information does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals.

This product had been tested in accordance with EN ISO 374-5:2016, and ISO 16604:2004 to meet the requirements for resistance to penetration by blood-borne pathogens, test method using Phi-X174 bacteriophage.

The gloves were found to meet with the REACH annex XVII requirements for Polycyclic Aromatic Hydrocarbons (PAHs).

Components used in glove manufacturing may cause allergic reactions to some users. If allergic reactions occur, seek for medical advice immediately.

In case a serious incident has occurred in relation to the device, inform Medeco BV as listed on the packaging, your local distributor and your local healthcare authority.

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H&R Healthcare Ltd.
Mediq Suisse AG
Medeco B.V.